



PREPARATION

GLUSHU

You can reshape the shoe as you would do with a normal aluminium horseshoe. *Good hoof prep is critical to the shoe remaining secure for the complete trim cycle 6-8 weeks.*

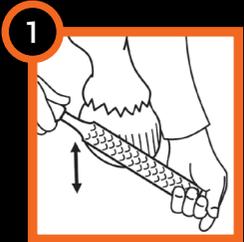

1 Trim and fine rasp the hoof as normal. Ensure all parts of the hoof where the Glushu will fit have been rasped and dust free (including the heels).

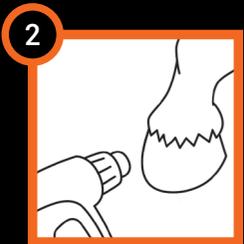

2 Make sure the hooves are dry. *Most glue failure is due to dirt or moisture in the hoof.*



3 When using Shufit Glue, express a small amount of glue until it is dark grey before using.


Make sure the glue is warm (25°C - 30°C | 77°F - 86°F).

REMOVAL

DRAWKNIFE



1 Using a draw knife, cut the cuff from the hoof using downward strokes along the glue line.

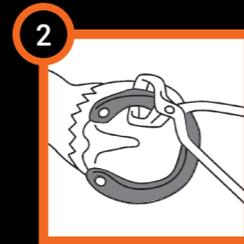

2 Using nippers, cut the metal shoe from the hoof.



3 Remove any remaining glue or cuff with a rasp or buffer.

PRE FIX APPLICATION VIDEO



STANDARD APPLICATION VIDEO



MODIFICATION VIDEO



GLUSHU

GLUE-ON HORSESHOE



A flexible cuff bonded to a specially designed aluminium horseshoe.

This gives the horse a light, hoof supporting, cushioned shoe which eliminates any stress or damage to the horse's hoof.

MORE INNOVATING PRODUCTS

CONTACT

GLUE-U® ADHESIVES

Glue-U Adhesives BV
Droogdokkeneiland 8
5026SR Tilburg
The Netherlands

+31 (0)13 545 31 18
info@glue-u.com

Glue-U USA Inc
215 Church Street
Georgetown, SC29440
USA

+1 (313) 509 - 7765
WWW.GLUE-U.COM



glue-u

adhesives



APPLICATION instructions

PRE FIX, STANDARD & WITH INSERTS



PRE FIX APPLICATION

PRE FIXATION METHOD

For horses who cannot hold their feet off the ground due to soreness. For glueing in severe cold circumstances.



1 Put a medium bead of Pre Fix CA Glue around the base of the hoof.



2 Spray the inside bottom of the shoe with the Pre Fix activator.




3 Holding back the side cuffs, aligning it with the frog and adjust to the correct position.

Hold for 30 seconds. Once the Pre Fix CA glue has set, place the hoof on the ground.

PRE FIX APPLICATION

PER PAIR OF GLUSHU'S


- 1x 150cc tube Shufit acrylic glue & mixingtips
- 1x Pre Fix Glue and activator spray (will last 6-10 pairs)
- 1x Hoof pick to help pull back the cuffs
- 1x Dispenser for shufit (universal with adapter)
- 1x Clear roll of hoofwrap

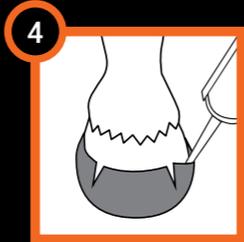

4 With the hoof on the ground, pull back the cuffs using a hoof pick and backfill inside the cuff with Shufit.



5 Use hoofwrap to keep the shoe in position while the glue is setting.

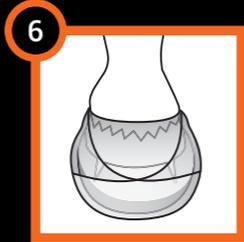

6 Place the hoof on the ground and leave for a minimum of 10 minutes before removing the hoof wrap.

STANDARD APPLICATION

PER PAIR OF GLUSHU'S

- 1x 150cc tube Shufit acrylic glue & mixingtips
- 1x Dispenser for shufit (universal with adapter)
- 1x Clear roll of hoofwrap



1 Put a medium bead of Shufit glue around the inside base of the shoe and cuff.



2 Holding back the side cuffs, slide the shoe onto the hoof, aligning it with the frog and adjust to the correct position.




3 Use hoofwrap to keep the shoe in position while the glue is setting.

Place the foot gently on the ground when the glue has set.

INSERTS APPLICATION

PER PAIR OF GLUSHU'S

- 1x Set GluShu inserts
- 1x Application set of your choice

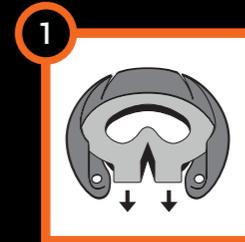

1 Apply the insert to the GluShu from above with the glue of your choice.

Glue-U CA glue: 10 seconds
Glue-U Shufit: 5 minutes

TIP! Fill with Glue-U pad material for the best support.

